


Transferring Licences

Information about transferring licences is in licensing section of Medmont Studio Help accessed from the Studio console > Help menu item and also in your users manual

Transferring Licences

The procedures to transfer Medmont Studio Licence(s) from one PC to another PC are as follows:


1. Install Medmont Studio on the machine that you wish to transfer the licences to.
2. When you click on the Medmont Studio icon, a prompt to run the licence manager should start automatically - Select yes to run the licence manager or Select [Tools->Licence Manager] to run the Licence Manager from within Medmont Studio if the prompt doesn't appear. This may occur if you are installing another component to an existing Medmont Studio installation and have reinstalled the software to add the component.
3. Click on the **Install Licence(s)** button.
4. Select **Transfer Licence from another machine via File** and click on the **Next** button.
5. Select the location for the licence transfer file which normally would be a floppy drive but could be over a network . Put a floppy disk in the A: drive and select the floppy drive (A:) location.

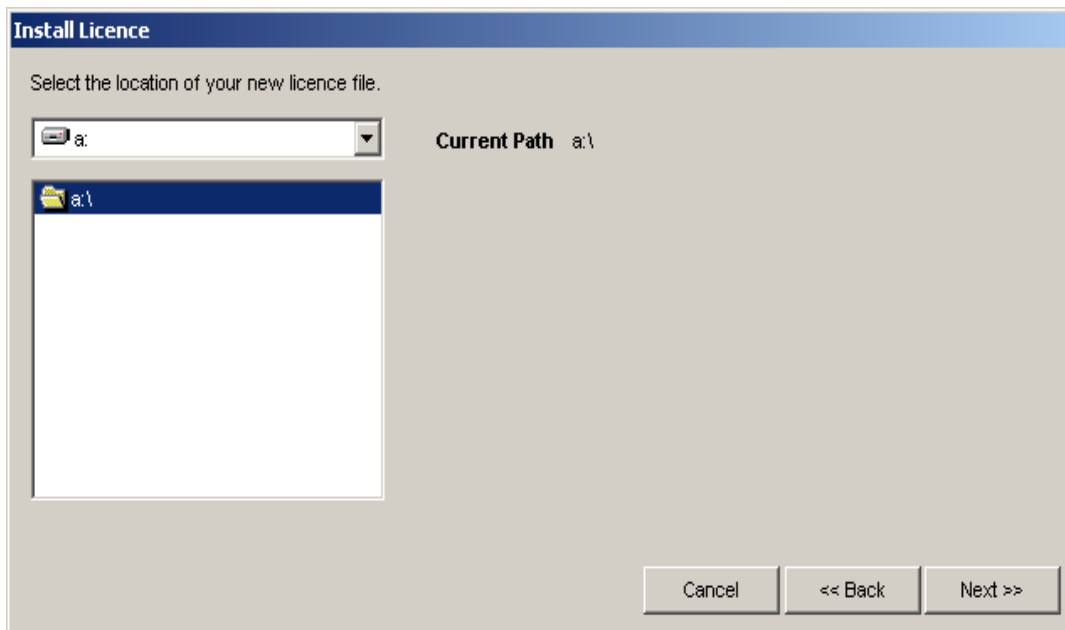

6. Select the **Next** button. This creates the licence transfer file in the specified location. Do not click on Finish yet! Take the floppy disk out.
7. Run Medmont Studio on the machine that currently has the licences installed on it.
8. Select [**Tools ->Licence Manager**] to run the Licence Manager.
9. Click on the **Transfer Licence(s)** button.

Transferring Licences

10. Select the licences you want to transfer from this machine by highlighting the licence's in the left list and clicking on the > button.

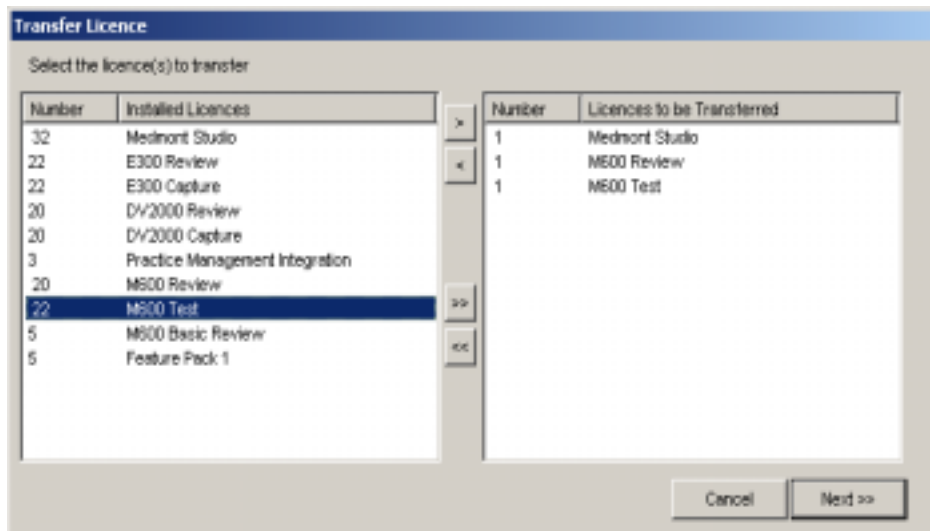

The licence(s) to transfer move to the right list under “licences to be transferred”

11. Click on the **Next** button.
12. Select **Transfer Keys via File** and then click on the **Next** button.
13. Put your floppy disk in The A: drive. From the prompt, “run the licence manager on the machine you wish to transfer the licence to” Select **Next**.
14. The location of the Transfer File should be A: Click **Finish**. This writes the Activation Key for the new PC to the specified file.
Take out the floppy disk from the machine that had the licence and return it to the machine that you wish to transfer the licence to.
15. Return to the PC that the licences are to be transferred to. If you are using a floppy disk to do the transfer, swap it back to this machine.
16. The dialog displayed in step 6 should still be displayed. Click on the **Finish** button to complete the transfer.
17. The licence's transferred appear now under “installed licences”.
18. **Close** the Licence Manager and Medmont Studio should now open.